# RelyX U200 AUTOMIX

Cemento de Resina Universal Autoadhesivo

## Descripción

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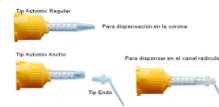
**3M Science. Applied to Life.™**

### Zirconia Crown Cementation

Simple Steps for 3M™ RelyX™ U200 Self-Adhesive Resin Cement

- Step 1 Prepare restoration:** Sandblast the restoration with aluminum oxide after finish (Mix 2 bar or 20 PSI, particle size 7-40µm).
- Step 2 Pre-treat tooth:** Remove provisional restoration. Mechanically clean prepared tooth (e.g. with pumice paste).
- Step 3 Apply cement:** Apply two clicks of the cement paste onto a moist tooth. Hand mix with a spatula until a homogeneous mix is obtained. Wipe the Clicker barrel independently to avoid contamination of the material and premature setting. Ensure that the tip is put firmly in place (clicking sound).
- Step 4 Seat and Clean-up:** Firmly seat the crown with finger pressure. Remove excess cement with a rubber bulb holding the crown in place.
- Step 5 Final cure:** Light cure for 30 seconds per surface or wait 6 minutes from start of mix for dark cure. Finish and polish as needed.

**Tip:** If manufacturing is done in laboratory before try-in, clean with isopropyl alcohol and all dry with oil-free air.

**Tip:** If manufacturing is done in laboratory before try-in, clean with isopropyl alcohol and all dry with oil-free air.

**Tip:** Mix any residue (temporary cement, desiccants, air-drying, etc.) is completely removed. Do not use H<sub>2</sub>O, EDTA or NaOCl.

**Tip:** For Automix Delivery: Discard a small amount of cement into the mixed to ensure a perfect mix. Discard this material.

**Tip:** For Clicker Delivery: Use the mixing spatula to apply the cement into the crown.

**Tip:** For Automix Delivery: Dispense cement directly into the crown.

**Tip:** Do not exceed recommended bond cure time. Excessive duration will be difficult. For a controlled curing time, use 3M™ Elite™ LED Curing Light or 3M™ Elite™ DeepCure™ LED Curing Light back curing function.

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